


## SARS-CoV-2 & Influenza A/B & RSV Antigen Combo Test Kit (Self-Test)

### Product Brochure

Version: V1.1-2022.07.05

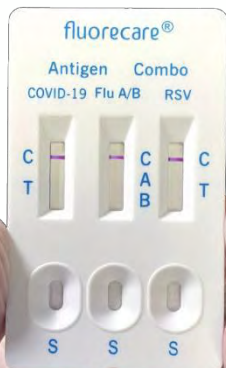

## Introduction

During the first few days of a Covid-19 illness, symptoms tend to be nonspecific and may include cough, fever and sore throat. These symptoms can be indistinguishable from other viral infections, such as influenza, respiratory syncytial virus (RSV), or viruses associated with the common cold. Because of this, combo diagnostic testing is needed and important, not only to determine the cause of the illness, but also to determine whether treatment is appropriate.

fluorecare<sup>®</sup> SARS-CoV-2 & Influenza A/B & RSV Antigen Combo Test Kit is an easy and rapid way to identify which infections cause the symptoms. It can be used everywhere and everytime by yourself.



**Efficiency: 4 in 1 test**



**Rapid: Results in 15 mins**



**Non-invasive diagnosis: Nasal swab sample**



**Convenient: Easy operation and no need for extra Equipment**



**Simple Storage: Room temperature (2-30°C)**



**Comprehensive: Effective for mutants of SARS-CoV-2 Alpha, Beta, Gamma, Delta and Omicron**

## Components



Test card

Tube

Test card bag

Swab

Introduction

## Characteristics

<b>Method</b>	Colloidal Gold
<b>Test Time</b>	15 mins
<b>Shelf Life</b>	18 months
<b>Sample Type</b>	Nasal swab
<b>Specification</b>	Kits for 1T, 2T, 5T
<b>Storage</b>	Room temperature (2-30°C)



## CE Certificate

# CeCert.

## CERTIFICATE

DIRECTIVE 98/79/EC  
EC DESIGN-EXAMINATION

CeCert Sp. z o.o. hereby confirms that manufactured by

**Shenzhen Microprofit Biotech Co., Ltd.**

Rm. 405, 406, Zone B/4F, Rm. 205, 206-1, 207, West Side  
of Zone B/ 2F, Haowei Building, No. 8 Langshan 2nd  
Road, Songpingshan, Songpingshan Community,  
Xili Street, Nanshan District, Shenzhen, P.R. China

*in vitro* diagnostic medical device for self-testing

**fluorecare SARS-CoV-2 & Influenza A/B  
& RSV Antigen Combo Test Kit**  
catalogue numbers: MF-71-1, MF-71-2, MF-71-5

in term of the design conforms to the requirements of Annex III  
section 6 to Directive 98/79/EC (as amended) implemented into Polish  
Law, as evidenced by the assessment conducted  
by CeCert Sp. z o.o.

CE 2934


Validity date: 12.05.2022 – 26.05.2025

Edition issue date: 18.05.2022

Check it



CeCert Sp. z o.o.  
ul. Żurawia 32/34  
00-515 Warszawa

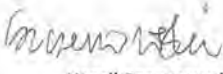  
Kamil Szczurowski  
Director of *in Vitro* Diagnostic Medical Device  
Certification Department

www.cecet.pl  
e-mail: [biuro@cecet.pl](mailto:biuro@cecet.pl)

Certificate no: CeCert/092/W/E.2

## Clinical Accuracy

### SARS-CoV-2

Method		RT-PCR		Total Results
fluorecare <sup>®</sup> SARS-CoV-2 & Influenza A/B & RSV Antigen Combo Test Kit	Results	Positive	Negative	
	Positive	342	0	342
	Negative	26	450	476
Total Results		368	450	818

Positive correct rate ( Clinical sensitivity )at Ct<38=92.93%(95%CI:89.82%~95.33%)

Negative correct rate (Clinical specificity) = 100% (95%CI:99.18% ~100%)

Method		RT-PCR		Total Results
By Lay Person	Results	Positive	Negative	
	Positive	30	0	30
	Negative	2	87	89
Total Results		32	87	119

## Clinical Accuracy

### Influenza A

Method		Reference Product		Total Results
fluorecare <sup>®</sup> SARS-CoV-2 & Influenza A/B & RSV Antigen Combo Test Kit	Results	Positive	Negative	
	Positive	104	0	104
	Negative	9	555	564
Total Results		113	555	668

Clinical sensitivity = 92.04% (95%CI:85.42% ~96.29%)

Clinical specificity = 100.00% (95%CI:99.34%~100.00%)

Method		Reference Product Professional Test		Total Results
Self-Test	Results	Positive	Negative	
	Positive	17	0	17
	Negative	0	102	102
Total Results		17	102	119

## Clinical Accuracy

### Influenza B

Method		Reference Product		Total Results
fluorecare <sup>®</sup> SARS-CoV-2 & Influenza A/B & RSV Antigen Combo Test Kit	Results	Positive	Negative	
	Positive	80	0	80
	Negative	8	580	588
Total Results		88	580	668

Clinical sensitivity = 90.91% (95%CI:82.87% ~95.99%)

Clinical specificity = 100.00% (95%CI:99.37%~100.00%)

Method		Reference Product Professional Test		Total Results
Self-Test	Results	Positive	Negative	
	Positive	11	0	11
	Negative	1	107	108
Total Results		12	107	119

## Clinical Accuracy

### RSV

Method		Reference Product		Total Results
fluorecare <sup>®</sup> SARS-CoV-2 & Influenza A/B & RSV Antigen Combo Test Kit	Results	Positive	Negative	
	Positive	63	0	63
	Negative	3	602	605
Total Results		66	602	668

Clinical sensitivity = 95.45% (95%CI:87.45% ~99.05%)

Clinical specificity = 100.00% (95%CI:99.39%~100.00%)

Method		Reference Product Professional Test		Total Results
Self-Test	Results	Positive	Negative	
	Positive	31	0	31
	Negative	1	87	88
Total Results		32	87	119



## About us

Founded in 2009, Microprofit Biotech is a national high-tech enterprise from China, which is specialized in medical devices and reagents in the field of in-vitro diagnosis (IVD) with the idea of POCT (point-of-care-testing). fluorecare®, the official brand of Microprofit Biotech, dedicated to entire products of point-of-care (POC) tests based on immunofluorescence quantitative assay platform.



With professional R&D, manufacture, marketing, sales and service team, Microprofit Biotech sold products to nearly 100 countries&areas and has won a good reputation around the world. Microprofit Biotech strictly complies with national and international regulations and standards and got many certification,such as CE certificates, FDA certificates, FSC certificates and ISO 13485:2016 quality system certification.



## Microprofit Biotech

### Shenzhen Microprofit Biotech Co.,Ltd.

Add: Haowei Building, No. 8 Langshan 2nd Road,  
Songpingshan, Songpingshan Community, Xili  
Street, Nanshan District, Shenzhen, P.R. China

Tel: +86-755-86325154, +86-755-61688835

Fax: +86-755-61688111

Websit: [www.microprofit-bio.com](http://www.microprofit-bio.com)

E- mail: [bio@microprofit-bio.com](mailto:bio@microprofit-bio.com)

