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EINFÜHRUNG

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1. Egan M.E., Lipski M.S., Diagnosis of Vaginitis, American Family Physician, September 2000, pp 1-14.
2. Sobel J.K. et. Al., Vulvovaginal candidiasis: Epidemiologic, diagnostic and therapeutic considerations. American Journal of Obstetrics and Gynecology V178(2), 1998.

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For self-testing

INTRODUCTION

VagiQUICK® is a qualitative rapid diagnostic self-test for the detection of *Candida* antigens in vaginal secretions sampled by a swab. The kit is to be used by women who suffer from vaginal symptoms like

- Acute itching
- Vaginal discharge, soreness, irritation
- Rash on outer lips of the vagina
- Genital burning that may increase during urination

The test is based on the immunochromatographic technology. The swab sample is added to the sample pad via the cap filled with dilution buffer.

From there the sample migrates into the conjugate pad, releasing anti-Candida antibodies conjugated to blue-colored latex beads. The newly formed Candida antigen-antibody-conjugate complex moves further across the nitrocellulose membrane strip. The formed complex is captured on the support membrane by immobilized anti-Candida antibodies, producing a signal line.

If Candida is present in the swab sample, there will be a biological reaction and the result will be positive. A blue line will appear in the test line region (T)


on the membrane. The test includes a built-in control system realized by a control line (C). The control line confirms the correct performance of the test.

To determine and interpret the results correctly, it is necessary to carefully read the instructions. All details should be understood before performing the test.

TEST CONTENTS

- 1 test cassette (Vaginal candida test) in a sealed pouch
- 1 sterile swab


Coplan Flock Technologies
Via F. Perotti, 16/18
25125 Brescia, Italy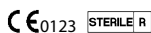

- 1 instructions for use

Additionally required:


- 1 timer

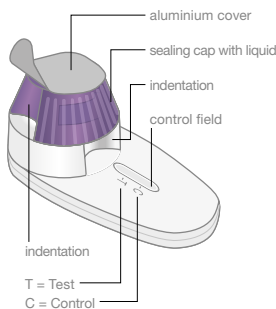1. Remove the device from the sealed pouch and place it on a flat horizontal surface.
2. Hold the device stable with one hand. Carefully remove the aluminum cover of the purple cap with the other hand. **Note! The purple cap contains liquid.**
3. Remove the swab from the wrapper.
4. Insert the swab one inch (2cm) inside the vagina and rotate for **20 seconds**. Pull the swab out carefully!
5. Rotate the swab in the liquid placed in the purple cap, for **20 seconds**.

6. Pull out the swab carefully while squeezing it against the inner wall of the purple cap. **Discard the swab.**
7. Hold the device stable with one hand. **Turn the purple cap counterclockwise until it stops (b), and then turn it back to starting point (a).**

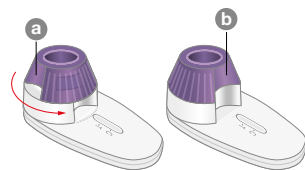8. Repeat step no.7 two more times. **In the final position the notches on the purple cap and the white base must be aligned (b).**
9. **Read results after 10 minutes.** A blue line should appear at the control (C) mark. In case of positive results, an additional blue line should appear at the test (T) mark. **In case of negative or unclear result read again after 10 more minutes.** Read the result after no more than 20 minutes because otherwise this can lead to false-positive results.

EVALUATION OF TEST RESULTS

For the interpretation of results, first determine whether there is a colored line in the test window at position (C). The intensity of the control line is of no significance.

POSITIVE TEST RESULT

If a faint light to dark blue Control line (C) is visible in the result window along with a light to dark blue Test line (T), the test result is positive.

The test result means that *Candida*-antigens are present in the vaginal secretion. You should consult a physician to confirm the result and start treatment.

NEGATIVE TEST RESULT

If **only a light to dark blue Control line (C)** is visible in the result window with **no Test line (T)**, the test result is negative.

The test result means that **no Candida-antigens are present in the vaginal secretion.**

In case of persisting symptoms or complaints, we advise you to consult a physician.

INVALID TEST RESULT

If both lines do NOT appear or only the TEST LINE (T) appears, no results can be derived from the test. You must perform a new test, while adhering strictly to the instructions.

WARNINGS AND IMPORTANT INFORMATION

- The test is not reusable.
- The test works only if the instructions are followed carefully.
- Negative test results do not eliminate other infections. Consult your physician!
- The test should not be used during the menstrual period.
- Only to be used by women 18 years and older
- Do not use by pregnant women.
- Do not use this test within 7 days after taking medications for vaginal infections.
- Do not use this test within 24 hours after using cream, gel, foam, douching solutions or any other vaginal product.
- Do not use after the expiry date stamped on the foil pouch.
- Store in a dry place at room temperature between 2°C to 30°C.
- Keep out of reach of children.
- For in-vitro diagnostic use only. Not to be taken internally.
- Dispose of the device and the swab in the regular trash after use.
- Do not use the swab if it was dropped.
- Do not use broken test components
- Do not use if you spilled liquid from the cap. The amount of the fluid in the cap is crucial for the correct function of the test.
- Do not rotate the cap before removing the seal or inserting the swab. Once the upper part of the device is rotated it is not reversible and the test is not valid.

BIOLOGICAL REFERENCE RANGE AND LITERATURE

VagiQUICK® can detect Candida-concentrations starting from 6,0x10³ cells/mL. The test has an accuracy of 90 %.

1. Egan M.E., Lipski M.S., Diagnosis of Vaginitis, American Family Physician, September 2000, pp 1-14.

2. Sobel J.K. et. Al., Vulvovaginal candidiasis: Epidemiologic, diagnostic and therapeutic considerations. American Journal of Obstetrics and Gynecology V178(2), 1998.

Please contact the manufacturer for further information and supplementary literature.

Explanation of symbols:

Follow instructions	In vitro diagnostic medical device (for external use)	Best before (See imprint on package)
Store at 2°C - 30°C Do not Freeze.	Content sufficient for 1 test	Do not reuse
Manufacturer	Sterilization by irradiation	Batchnumber (See imprint on package)
Catalogue number		

TEST PREPARATION

Let test cassette stand at room temperature (15°C to 27°C) before performing the test.

TEST PERFORMANCE

Do not open this test kit until you are ready to use it. Do not use the test if the pouch is damaged. Make sure that you have a watch available. Make sure to perform the test on a clean surface, wash your hands! The test may be performed at any time of the day.

CE 0483

Instructions English
Revision from 2018-11 (Rev. 03)

REF 500000

NanoRepro AG
Untergasse 8
D-35037 Marburg
www.nano.ag

For the love of life.