



COVID-19 + Flu A&B + RSV Antigen Combo Rapid Test

One Sampling, Four Results


Original Design, User-friendly

Easy to Use, Fast Results

Minimal Sample Needed (3 Drops)

Innovative Multiplex Respiratory Virus Rapid Test

Convenient Procedure for IRCFN-MC84

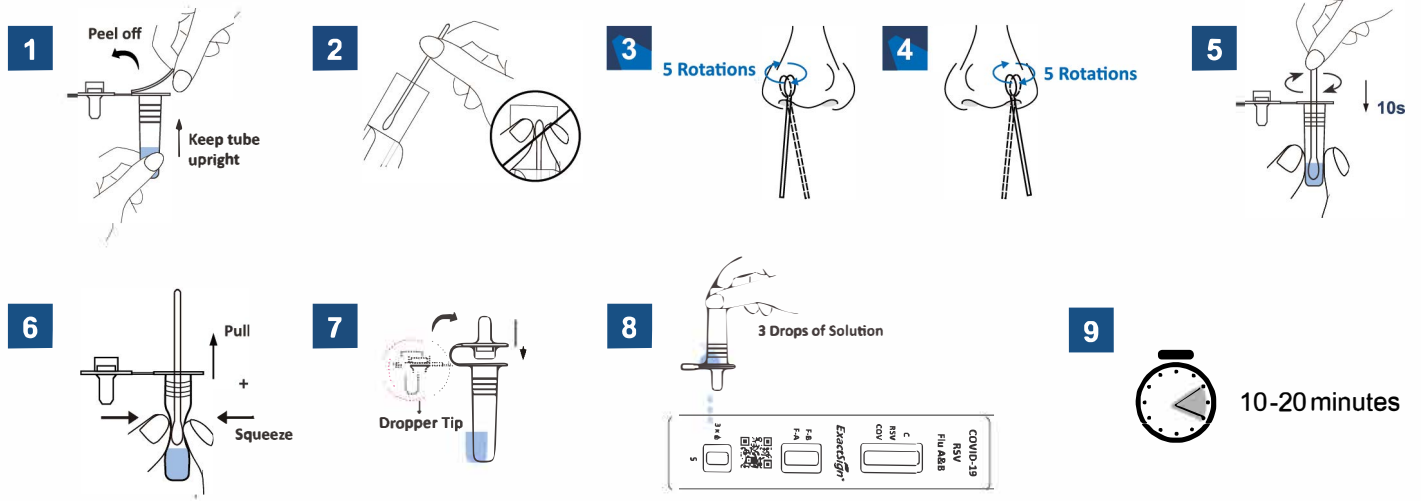

CONTROL SWAB



POSITIVE CONTROL



NEGATIVE CONTROL

RESULT INTERPRETATION

Positive (+)



Control Line

One colored line appears in the control line region (C).



XXX*Test Line

Colored line(s) appears in the T(XXX) region. A positive result in the T(XXX) region indicates that XXX was detected in the specimen. The shade of the colored line(s) in the T(XXX) region may vary. The result should be considered positive whenever there is even a faint line.

*** means the test line region of T(RSV), T(COV), T(F-B) and/or T(F-A).

Negative (-)



Control Line

One colored line appears in the control line region (C).



XXX Test Line

No line appears in the T(XXX) region. A negative result indicates that RSV, COVID-19, Influenza A&B antigen are not present in the specimen, or is present below the detectable level of the test.

Invalid



Control Line

Control line fails to appear.

Ordering Information

Cat.No.	Product	Specimen	Format	Pack
IRCFN-MC84	COVID-19+Flu A&B+RSV Antigen Combo Rapid Test Cassette	Nasal Swab	Cassette	25 T
IRCF-MC84	COVID-19+Flu A&B+RSV Antigen Combo Rapid Test Cassette	Nasopharyngeal Swab	Cassette	25 T